

## DIRECTIONS FOR USE:

1. Open the plastic package and remove the weeSTRAPS™. Discard the plastic package.
2. Attach straps in the correct position on the board so that when the legs and arms are secured, the foam side of the strap will be in contact with the skin or a protective cloth.
3. When all straps are correctly placed, position the board on a flat, immobile surface. Attach straps to the board BEFORE placing the child on the board.
4. Position child carefully on the board.
5. To secure child's arms and legs, position the straps as close as possible to the elbows and knees. Fasten straps by pulling the end of the strap with the tab over the end of the strap without the tab. Gently press down on the tab to attach. Gently hold the arm or leg against the board while fastening the tabs. The straps should fit snugly.

## REMOVAL AND DISPOSAL:

1. To unfasten straps, pull tab away from the strap. After the child has been removed from the board, pull the straps from the underside of the board to remove the straps from the board.
2. Dispose of straps according to institution policy.

# weeMEDICAL weeSTRAPS™

## BABY BOARD STRAPS

Contents: Four 11.4" (29 cm) Polyurethane  
Foam/Polyester Fleece Tricot Straps



single use only

### INTENDED USE:

Used to secure the position of the infant during minor surgical procedures (e.g. circumcision) or treatments (e.g. X-RAY, I.V. insertions). For use with baby positioning boards only, such as the weeSECURE™ baby board.



latex free

### PRECAUTIONS:

**R<sub>x</sub>**  
**ONLY**

- US Federal Law restricts this device to sale by or on the order of a physician or licensed practitioner.
- For single patient use only. Discard after use.
- weeSTRAPS™ are not to be used as protective restraints.
- Never leave child unattended while using the straps or board.
- Never carry or lift the board while the child is on the board.
- Avoid positioning straps over peripheral I.V. sites, dressings, or wounds.